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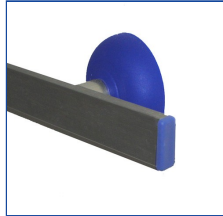
Wandschiene 25 x 10 mm Edelstahl

Instructions for Assembling and Use

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1. General information / intended use:

The wall rail is ONLY designed to carry/hold medical equipment that has the exact fitting attachment possibilities for wall rail 25 x 10 mm (e.g. IV-Poles, etc.). The wall rail is designed to mount ONLY to a strong and corresponding wall construction.

2. Assembling / Mounting

1. Please mark the fastening position on the wall and drill the holes (1) To mark the mounting points, the rail should be first aligned in the right position at the wall and then needs to be pushed against the wall with a slightly beating on the rail. We recommend to use a rubber hammer or other adequate tool in order not to damage the surface of the rail.
2. Put the pegs (2, not included in the standard) into the drilled holes.

Attention! Please use the corresponding and special pegs in case of plaster board walls!

3. Pre-assemble the spacers (6) with the wall plates (4) and adequate screws (3, not included in the standard). (Spacers will stay turnable). Recommended screw for concrete wall construction: DIN571 M6x50
4. Wall rail (7) can be plugged in to the control of the intervals. Screw screws tightly. Remove wall rail once again.
5. Put the cover cap (5) over the spacer.
6. Plug in wall rail (7) together with the spacer and screw tightly the stud bolts (8).
7. Connect grounding to the already specified position on the wall rail disc (every rail 1 piece). The eyelet (the sizes are about 2,5-3mm² and 4-6mm²) needs to be screwed together with the disc and the screws on the wall plate (See Picture 2).
The drilling (B) is already designed to the implementation of the EP cable.
8. Push cover cap (5) against the wall.
9. Insert end caps into sides of wall rail 25 x 10 mm.

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The load capacity of the wall or wall construction has to be selected in a way that the forces of the rail can be safely supported. The relevant safety factors have to be taken into consideration, when statics are verified. If needed, detailed static considerations have to be made.

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
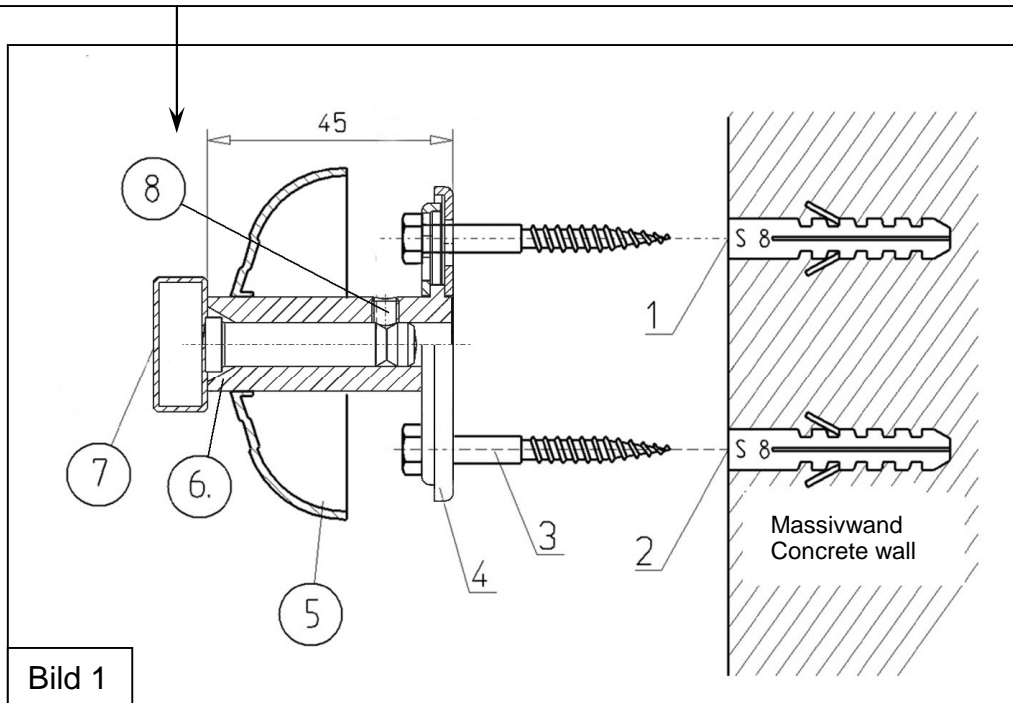
Max. load capacity for 1 meter wall rail – when 3 fastening points are equally spread: 100 kg

Attention! The max. load capacity is only valid for the wall rail itself. Please do consult your structural engineer in order to get an advice on what is the max. load capacity of your wall.

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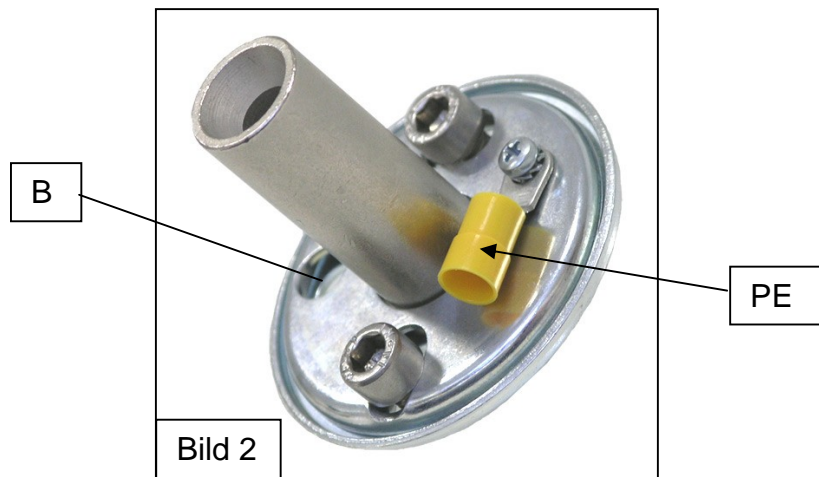Assembling example for a concrete wall.

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The mounting of the Wall Rail is properly accomplished when:

- the Bush is aligned with the fittings of the wall rail.
- ensures that the thread screw engages into the groove of the wall rail bolts (thread screws should not stick out of the bush's thread).
- the rosette shows no damages and to cover the thread screws as safety.

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3. Cleaning and disinfection

You can clean the system using a mild detergent. For the subsequent disinfection, you can use standard commercial disinfectants available such as Amocid, Baktolan or Sagrotan.

As there are so many different cleaning and disinfection products on the market, we cannot offer any further recommendations as we are not aware of their potential reactions with materials used and so cannot rule out the possibility of damage.

4. Inspection / maintenance

The system must be inspected every six months to make sure all components are functioning properly. Please check the following points:

- Are the screws (4) properly connected/screwed to the wall?
- Is the wall rail itself properly connected to the spacers (6)?

To be tested as followed:

- ✓ the Bush is aligned with the fittings of the wall rail.
- ✓ ensures that the thread screw engages into the groove of the wall rail bolts (thread screws should not stick out of the bush's thread).
- ✓ the rosette shows no damages and to cover the thread screws as safety.
- Are the plastic caps (5) properly put into the wall rail (at the endings of the wall rail) – risk of injure?
- Is the grounding still connected properly?

Additionally to the a.m. inspections guidelines, the current existing "Operator's ordinance" has to be fulfilled.

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