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Infusionsständer I611

Nach EN 60601-1 

Directions for assembly and use

IV-Pole I611

according to EN 60601-1 

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1. General / purpose:

- The IV-Pole series 611... with special screwed height adjustment is an IV-Pole for gravity infusions.
- To ensure that the comprehensive safety demands on spaces used for medical purposes are met to your satisfaction, **the safety instructions have to be heeded.**

2. Safety instructions:

- It is absolutely necessary when setting up, handling and using the device that these instructions are followed precisely.
- The devices are only to be used for the purposes named in the directions for use. Any other use may conceal unknown hazards and is to be refrained from.
- The allowed maximum weight per bottle-hook is 2,0 kg.
- Assurance against tipping (standing assurance) is guaranteed for up to 10% (5,7°) inclination for a load of 8 kg. Do not use or dispose the IV-pole on inclined support!
- Heavy objects like lead aprons etc. are not to be hung on the IV-Pole.
- When moving, make sure that the roller-brakes have been unlocked. The IV-Pole has to be moved by hand only! Rolling across obstacles, soft flooring and slants must be done with particular care. If the slope is more than 10%, then the inner tube has to be at shortest / min. position.
- Pumps and other devices must be mounted to the lowest third of the IV-Pole. Please make sure, that the max. load capacity (15kg) will not be exceeded.

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3. Assembling

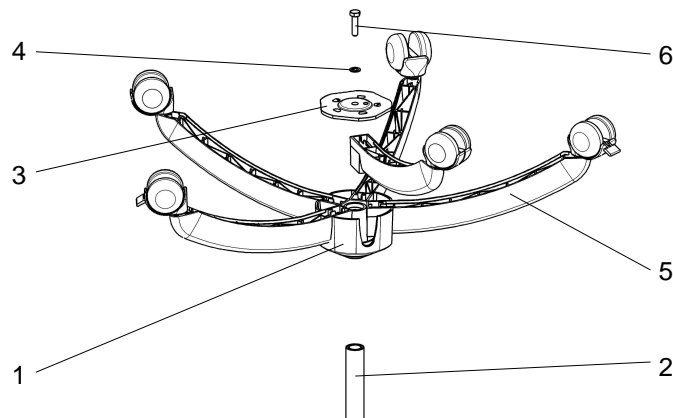
- **Attention:** Do not open the screw for height adjustment (12) without holding the inner tube (11) because the load might slide down uncontrolled.
- The height adjustment screw has to be tightened only as much as the load is held tight secure. Overwinding the screw might damage the IV-Pole.
- Infusion accessories may only be used on IV-Poles manufactured expressly for that purpose, as otherwise safety from tipping is no longer guaranteed.
- IV-Poles not in use are always to be locked with the lock rollers.
- The electrically conductive rollers serve only to divert static charges.
- The person who is selling the product, is only responsible for the safety and proper functioning of the IV-Poles when the directions for assembly and use have been followed in all points and only when original accessories are used.

Box contents:

Pos.	Qty	Name
1	1	plastic mid-section
2,11,12	1	IV-Pole
3	1	clamp disk with grooves
4	1	toothed disk
5	2	feet with locking rollers (antistatic)
5	3	feet with roller
6	1	hexagon head screw M8
7	1	plastic bottle-holder bottom section
8	1	plastic bottle-holder top section
9	1	glass-holder
10	1	drip glass
	1	screw-wrench SW13
	1	Directions for assembly and use

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3.1 Assembling the tripod foot:

- Slip the plastic mid-section (1) onto the Ø25 tube (2). The tightness enables a movement-free seating.
- Slip the feet (5) into the plastic mid-section.
- Take care that the two locking rollers are not placed near each other.
- Fit the clamp disk (3) with grooves facing inward to the plastic mid-section.
- Every groove of the clamp disk needs to be placed in the middle of one foot.
- The feet will be held by the grooves of the clamp disk.
- Tighten the clamp disk with the toothed disk (4) and the screw M8 (6) (ca. 15 Nm)

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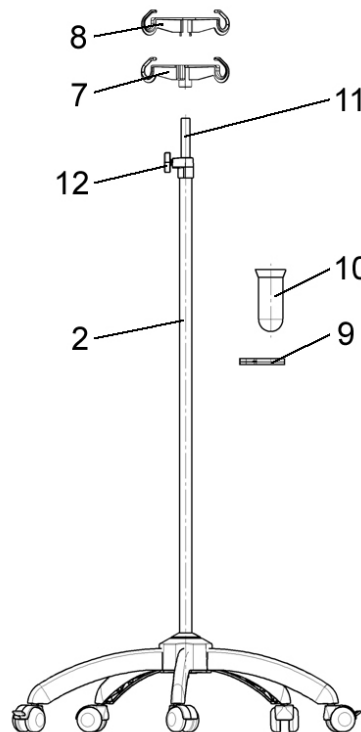
3.2 Assembling the plastic bottle holder:

- Slip the bottom part of the bottle-hook (7) onto the Ø18 inner tube (11).
- By hammering lightly (plastic hammer), bring the bottom part of the bottle-hook to it's end position.
- The tightness enables a movement-free seating.
- Slip the top part of the bottle-hook (8) into the t-grooves of the bottom part until the two barbs visibly snap into place.

ATTENTION:

When assembling the bottle-holder (top and bottom section), hold the Ø18 inner tube (11).

Installing the drip glass:

Clip the glass-holder (9) onto the Ø25 tripod tube. Insert the drip glass (10) into the glass-holder.

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4. Cleaning and Disinfection:

The infusion stand can be treated with mild cleaning agents, for the disinfection you should use commercially available disinfection agents for example: Amocid, Baktolan, Sagrotan.

5. Maintenance and Repair:

The infusion stand should be checked at least once a year for proper functionality of all important components:

Clamp-function for the inner tube Ø18 still dequate. To assure functionality of the locking and antistatic rollers clean them up.

Repairs have to be done by authorized personal or customer service with original equipment only.

6 Ambient conditions:

For transport and stock-keeping up to 3 months, the following stock-keeping conditions have to be heeded:

Temperature:	-25 bis 70°C
Moisture:	30-75%
atmospheric pressure:	700-1060 hPa

At application you have to assure the ambient temperature from 10°C to 75°C.

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