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Infusions- Deckenschienensystem

ICS 2 

Instructions for Assembling and Use

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Dear user,

These instructions were set up for the medical and technical staff in your facility.

The Infusion-Ceiling-Systems ICS 1 and ICS 2 are manufactured in accordance with current technical standards and safe for use. Nevertheless both systems can cause danger, being used by minor skilled people, or being used not in accordance in the proposed and written places of use.

Changes and displacements are not allowed for safety reason.

In case of problems that are not described in these instructions, please contact the manufacturer.

Please deposit these instructions next to the system, in order to make you find important information and safety-instructions

1. General information / intended use:

- The system is designed to carry IV bottles, infusion apparatus and other equipment, provided they have compatible fixings. It is also designed to carry provita accessories. To meet the requirements of specific applications, we offer a range of poles, tracks and accessories which can be combined as described in the product literature.
- The system is designed so that the poles, and therefore the apparatus and accessories they carry, can be positioned as required. This is achieved via the ceiling tracks and the rollers that run inside the tracks. The system is designed first and foremost for use in hospitals.
- The system is designed for use in intensive care and adjoining rooms.
- The ceiling tracks are securely attached to the structural ceiling via suitable sub-constructions.
- In order to fulfil the extensive safety-instructions for use of products in medical rooms, **you have to read the following safety-instructions.**

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2. Safety instructions

Before using the system, the user must check that the system is in good order and functioning correctly, and read the operating instructions and safety information carefully.

The unit is only designed for the purposes described in the operating instructions. Refrain from using the system for any other purpose, as this may present unforeseen dangers.

- The ceiling mounting must be structurally sound.
- The electrical installations in the room where the system is to be used must conform to the requirements of current regulations.
(Federal Republic of Germany VDE 0100-710).
- All extensions or modifications must be carried out by suitably qualified personnel and only using original provita components.
- The system must be installed by suitably qualified personnel in the manner described in the installation instructions.
- The poles and system apparatus must not be positioned directly above staff or patients.
- Apparatus should be attached to the poles in such a way that the poles are hanging as near vertical as possible.
- Do not use force when moving the poles.
- Do not exceed the maximum load of 2 kg for the individual bottle hooks.
- The Series I 100 poles can carry a maximum load of 15 kg. Make sure that the outer tube is not dented while installing accessories (this will adversely affect proper functioning).
- The maximum loading capacity of each carrier is 40kg. By the Duorail System (2 carriers) there are 80kg admissible. The extensible inner tube may be charged with max. 15 kg. The max. payload is about 30 kg by Monorail, respectively 60 kg by Duorail. The sliding hooks I100 – I102 may be charged with max. 15 kg. The admissible chargeability is reduced by 50% in case of 24 hours continuous operation.
- It is not allowed to overcharge the ceiling rail with more than 80 kg / 80 cm..
- When moving the poles please take note of the following:
 - The brakes have to be released.
 - Make sure that there is nothing in your way.
 - Please take care, that no cables or pipes can be disturbed, while moving the pole.

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3. Mounting

Please read and mount the system in accordance with the following instructions and safety information. The safety for people depends a lot on the proper installation of the system. Make sure that the ceiling is in accordance with the requested statics. If there are any questions, please contact an architect.

The kind of ceiling construction determines the kind of the wall plug. Please make sure that you fulfil all regional construction and building laws.

Make sure that the rails and the poles don't get into touch with any dust and dirt at the installation site.

We recommend the following order of assembling:

- Assemble the provita spacer (for intermediate ceiling in accordance with the drawings (Screw on everything a little bit)
- Mount the rail pieces to each other by the use of the connection pins and connection elements (see drawing)
- Mount the spacers to the concrete ceiling and make sure that they are in proper vertical and horizontal position. It might be helpful to mount the rail to the spacers, before mounting the spacers to the concrete ceiling. The holes in the rail (for mounting to the spacers have to be made when installing the system.

ATTENTION: Make sure that the internal surfaces of the rail don't get damaged while installing the rail.

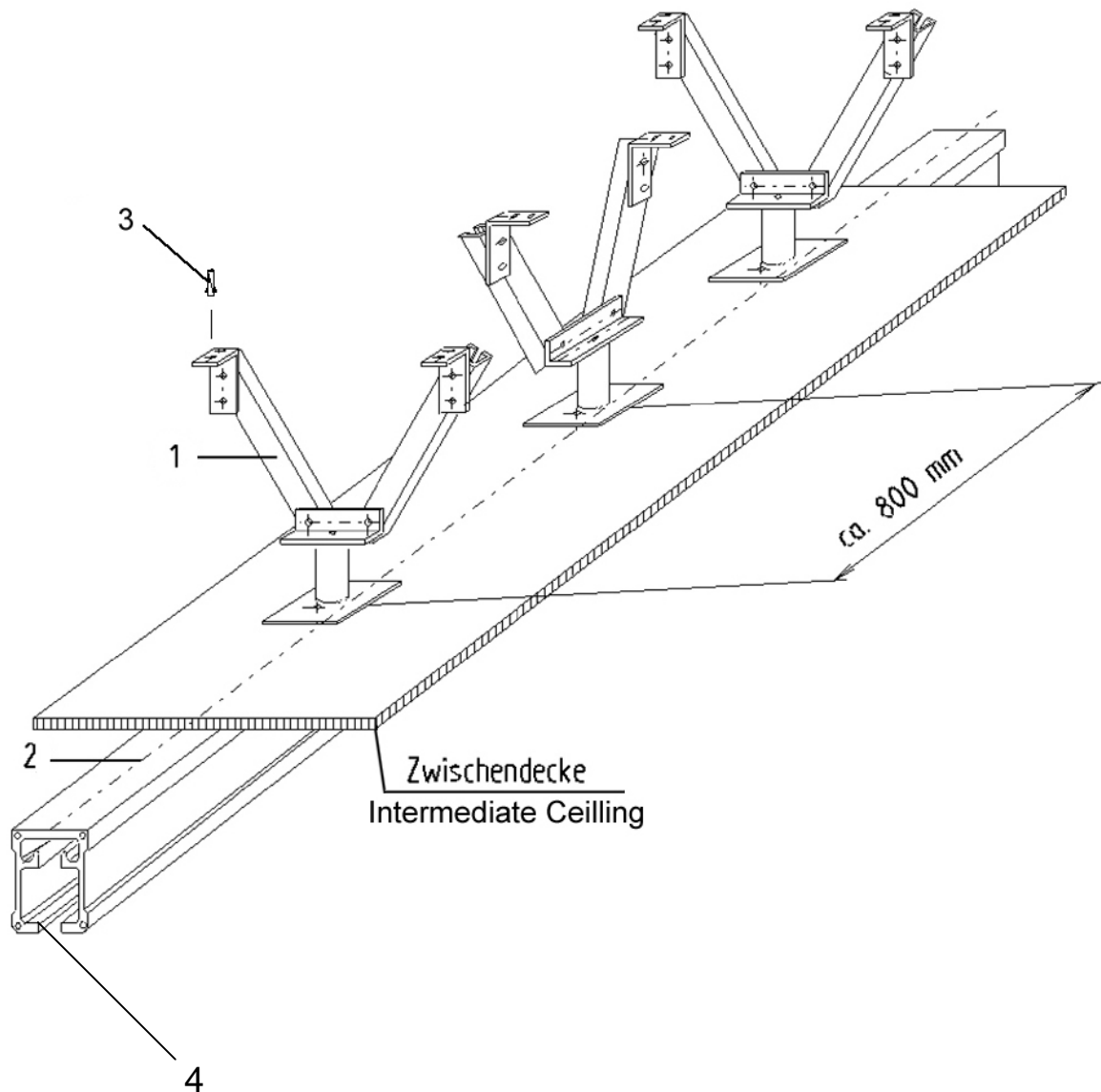
- In order to help you installing the rails as comfortable as possible, we recommend telescopic ceiling poles.
- **It is very important, that the rail is mounted horizontally and that there are no stresses.** After adjustment of everything, all screws can be tightened up.
- When mounting the intermediate ceiling, it might be needed to unassemble the rail again. The points of installation (where the spacers are positioned) have to be visible.
- **After proper installation of the rail, the internal running surfaces of the rails have to be cleaned carefully and some silicon spray has to be put on the same. There should not be any damages inside of the rail.**
- Please put now the poles via sliding hook or with carrier into the rail. Doing this, please check both again visually, whether all connections and screws are tighten up properly. **The carrier and the sliding hooks have to be free of any dust and dirt.**
- Please check the running characteristics of the poles and also the function of the poles.
- Do adjust the bottle hooks now and fix it with a locknut (Hooks opening showing to the top).
- Set up the cable carriers in to the rail.
- Now do assemble the end caps and end pieces.

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3.1 Mounting of the spacers (for intermediate/false ceiling)

- For the stability reason the spacers (1) have to be mounted 90° turned (see drawing), sidewise and lengthwise (2).
- The distance between the spacers shall not be more than 800 mm.
- The minimum „**tensile loading**“ ($F_{min.}$) of every **wall plug** (3) shouldn't be less than 80daN.
- After assembling the spacers, please clean the running surfaces (4) carefully.
- Mounting the U-System spacers see page 5.

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
3.2 Mounting of Connections between 2 rails

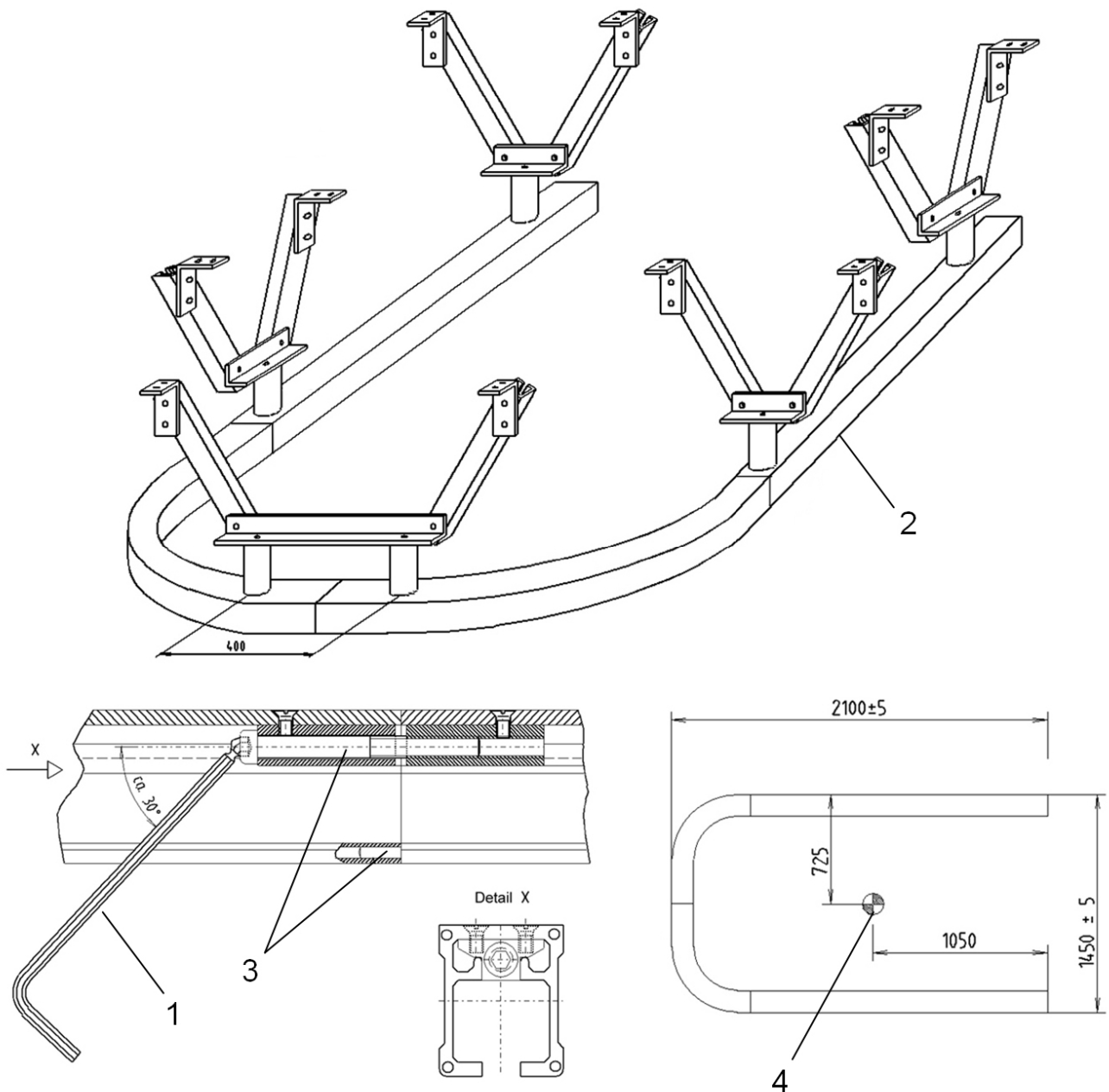- Combine the 2 rails with each other and screw them together with screw M8x80. We recommend a hexagon wrench key with ball-shaped head for screwing (1) (SW6).
- **ATTENTION!** Same numbers on the backside have to be assembled to each other (e.g.: 1 to 1, 1.1 to 1.1 etc.)
- Connector and connection pins (3) are pre-assembled by provita
- The U-shaped rail consists of two 90° curves and two straight rails. The position of the five spacers should be made as shown on the picture
- (4) shows the best position of the grounding outlet in the room

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
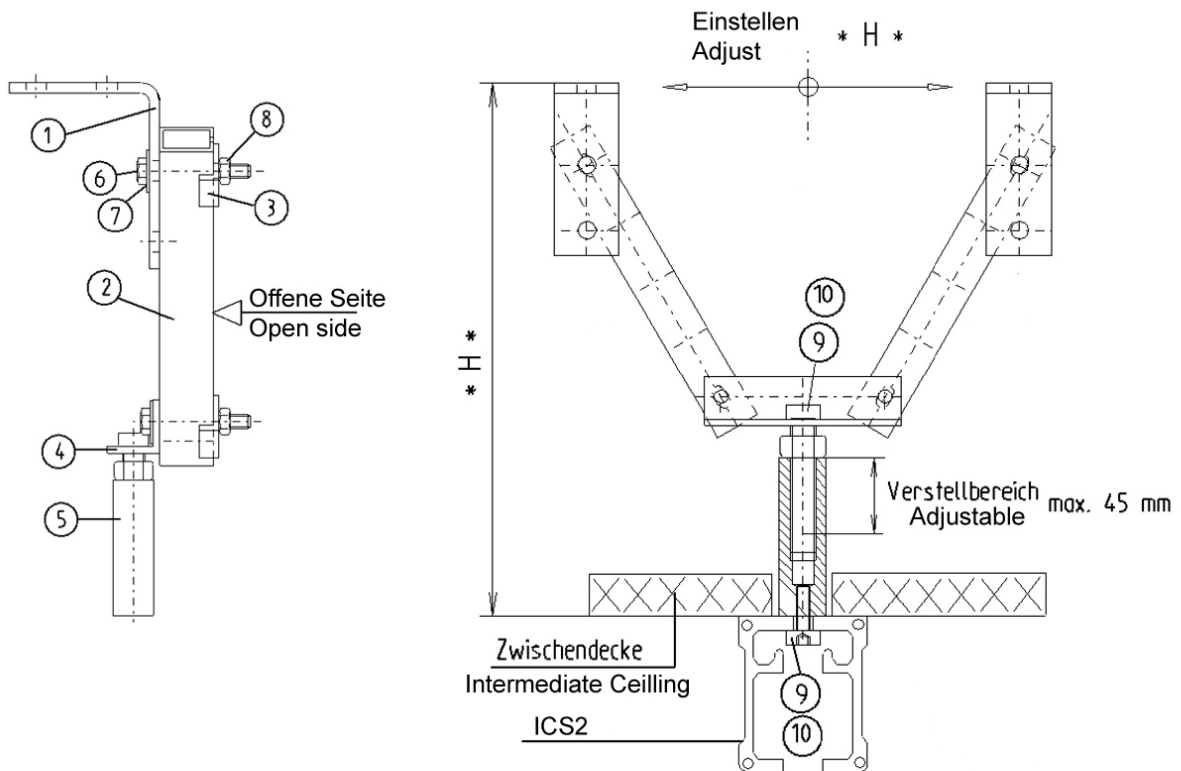
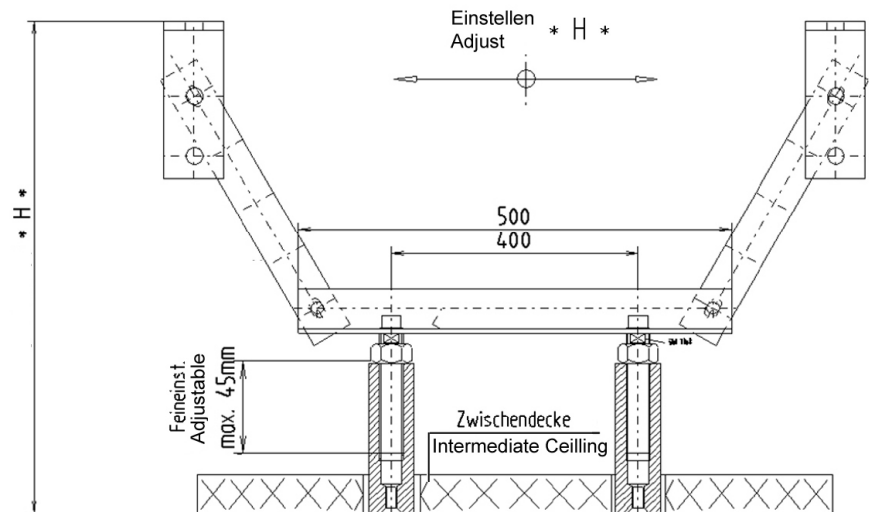
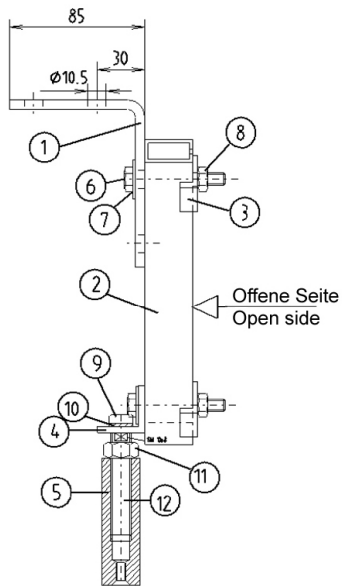
3.4 Assembling the end cap and the end piece

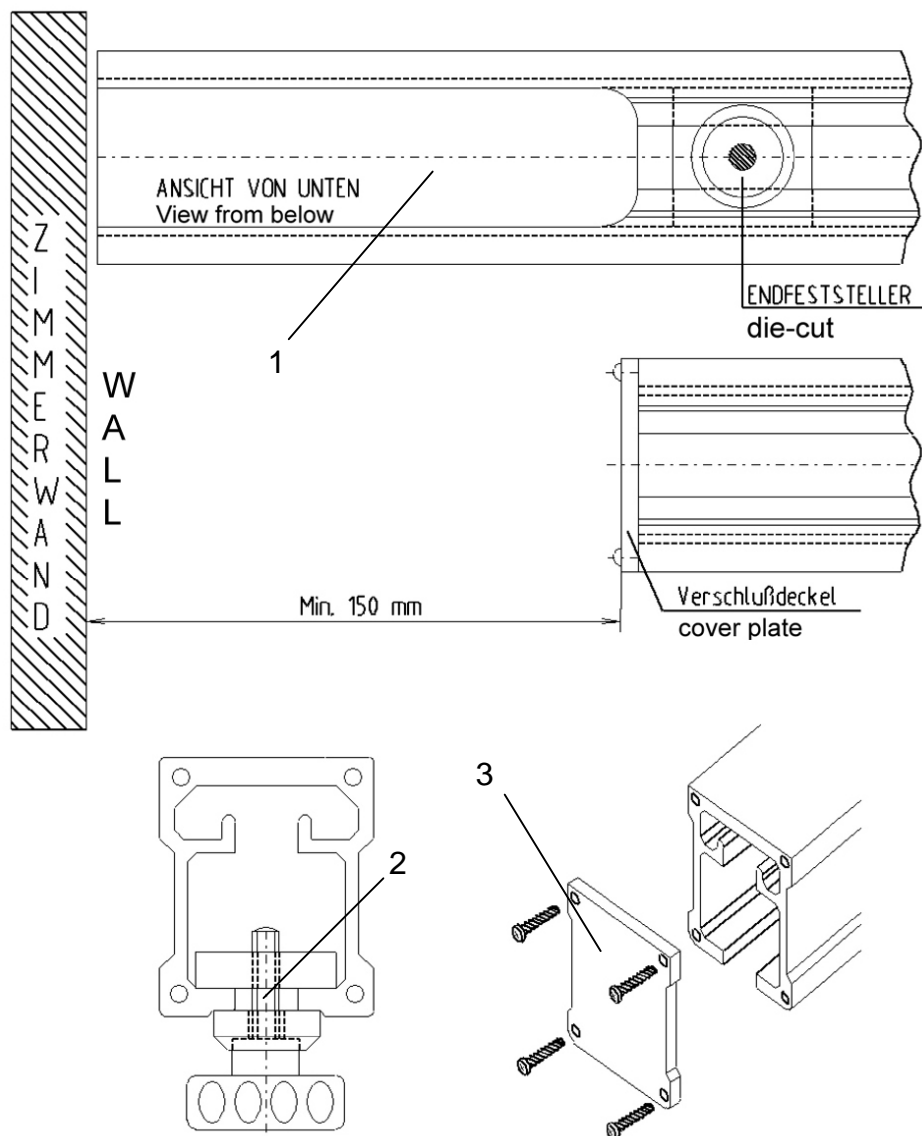- A die-cut (1) can be machined into the rail at the one end. Even though the rail is ending exactly at the walls, the carriers and sliding hooks can still be put into the rail.
- In case of using a die-cut, you have to use an end piece, too (2). It prevents the carrier or sliding hook from falling off the rail.
- Using further end-pieces you can set up special mobility frames.
- Without using a die-cut you should assemble the rail 150mm clear from the walls.
- Having enough room between end of the rail and the wall, you can use an end-cap (3) for closing the rail.

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3.5 Adjustment of brake force from roller trolleys

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- Unscrew stud bolt M3 (A)
- Turn the adjusting nut (B), with the socket wrench provided until the desired brake force is attained. See (F) for rotational direction.
- Tighten the stud bolt (A)

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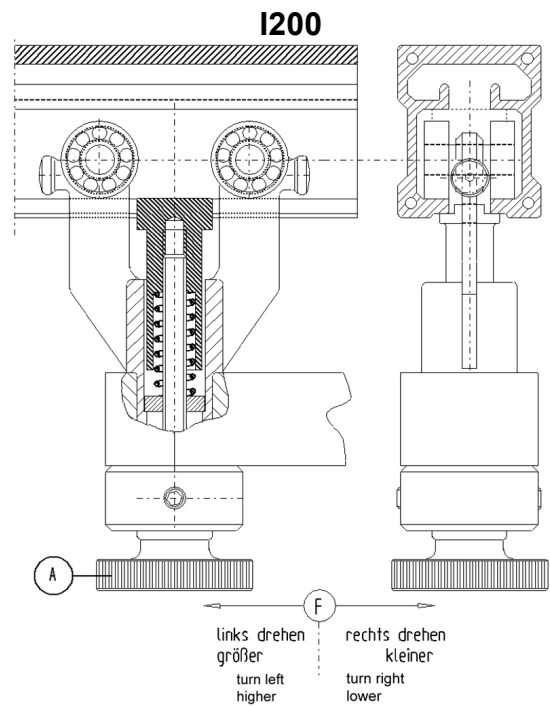
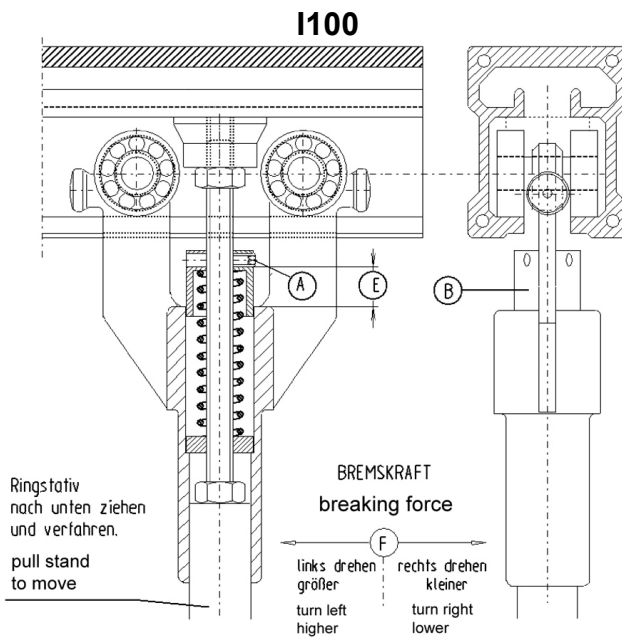
Attention! The spring resistance must be larger than the rod load, because otherwise the braking effect would not be sufficient. The maximum adjustment range of the adjusting nut is approx. 13 mm (E), and results in a brake force of 18 daN.

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- The round screw (A) at the bottom of the carrier can be turned like a screw.
- The braking force gets adjusted by turning (F).

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4. Cleaning and disinfection

You can clean the system using a mild detergent. For the subsequent disinfection, you can use standard commercially available disinfectants such as Amocid, Baktolan or Sagrotan.

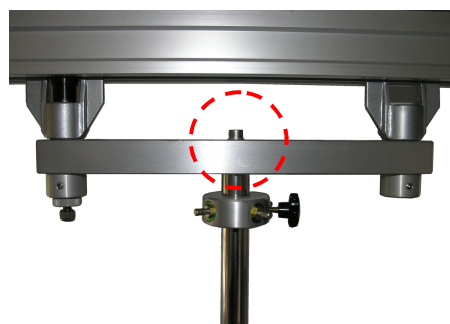
As there are so many different cleaning and disinfection products on the market, we cannot offer any further recommendations as we are not aware of their potential reactions with materials used and so cannot rule out the possibility of damage.

5. Operational Reliability Controls

The system has to be inspected every year to make sure all components are functioning properly. Please check the following:

- It has to be paid regard especially to the connection between the IV pole and carrier. Is the threaded connection (see image below) in order? The pole has to be fixed with no possible movement.
- Are the poles functioning properly? Is there enough braking force?
- Check the bottle hooks for damage.
- Position of the bottle hooks: are the hooks facing upwards and securely tightened?
- Check that the running surfaces in the track are clean.
- Visual inspection of the connection between pole and carrier or pole and sliding hook. Are all connections secure or has something come loose?
- Are components such as pump holders and accessories securely attached to the system?.
- Visual check of cables and electrical connections.
- Are the components positioned in such a way that the poles are hanging as near vertical as possible?
- Are the lower end stoppers on the poles functioning properly?
- Inform personnel working with the system about its functions and safety aspects.

Above all it has to comply with the regulations of the "Ordinance on the Operation and Use of Medical Devices".

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provita medical GmbH